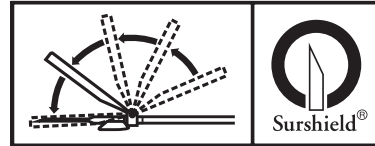
SV48E0252C

 **TERUMO**[®]

SURFLO[®]

CE
0123

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Winged infusion set with needle protection

ENGLISH

STERILE EO

Sterilized using ethylene oxide

Do not reuse

Consult instructions for use

Non-Toxic /
Non-Pyrogenic

Do not use if package is damaged

Manufacturer

EC REP

Authorised representative in the european community

INDICATION FOR USE

The Surflo Winged Infusion Set with Needle Protection (Surshield) is intended to access the peripheral vascular system, for intravenous administration of fluids and/or withdrawal of blood specimens using a syringe, luer adapter, or other compatible/appropriate devices. Additionally, after withdrawal of the needle from the patient's vein, the shield cover can be manually activated to cover the needle to minimize risk of accidental needle stick.

The device has no components made of natural rubber latex.

REPORT NEEDLESTICKS IMMEDIATELY AND FOLLOW ESTABLISHED PROTOCOL.

PRECAUTIONS

- Do not use if unit package is damaged.
- Use immediately after opening the unit package.
- For single use only.
- After use, secure the needle into the safety shield according to the instructions for use.
- Dispose of safely after single use to avoid risk of infection.
- Do not store at extreme temperatures and humidity.

CAUTION

Rx Only Federal Law (U.S.A.) restricts this device to sale by or on the order of a physician.

INSTRUCTIONS FOR USE

INSTRUCTIONS FOR USE: IV ADMINISTRATION

Aseptic technique, proper skin preparation and continued protection of the site are essential. Observe Universal Precautions on ALL patients.

PRECAUTION

Keep hands behind the needle at all times during use and disposal.

SET UP

1. Open the package.
2. Connect an administration set, syringe or etc. and secure.
3. Proceed with priming and confirm that administration fluid comes from the needle.

VENIPUNCTURE AND ADMINISTRATION

4. Flip the safety shield back away from the needle toward the tubing. Grasp wings.
5. Remove the needle protector. Caution: Care should be taken not to touch the needle.
6. Perform venipuncture and confirm proper positioning in the vessel (Fig. 1).
7. Secure the position of winged infusion set per facility protocol.

AFTER USE

8. Remove tape, if present, from wings.
9. Flip the safety shield forward towards the needle (Fig. 2). Grasp the wing with your thumb on top of the shield and your index finger under the wing. Completely remove the needle from the puncture site and apply digital pressure to the site using a sterile gauze pad (Fig. 3).

10. While maintaining the same grip on the device (per step 9 above), activate the safety measure by performing either of the following methods:
 - Recommended (one-handed) method:
(Reference OSHA 1910.1030 (d)(2)(vii)(B))
Slide your thumb forward over the shield while pinching your finger and thumb together (or press the safety shield against a hard surface such as a bedside table) until an audible click is heard and the safety feature is visually confirmed to be activated (Fig. 4a).
 - OR
 - Alternate (two-handed) method:
While keeping both hands behind the needle, grasp one of the wing tabs with your free hand to stabilize the device. (If necessary, reposition your original grip per step 9 being careful not to press against an exposed area of the needle.) Then, slide your thumb forward over the shield while pinching your finger and thumb together until an audible click is heard and the safety feature is visually confirmed to be activated (Fig. 4b).
- Caution: Needle must be carefully and firmly placed in the lock position of the safety shield.
11. Do not attempt to deactivate the safety device by separating the needle from the safety shield.
12. Dispose of used needles and materials following the policies and procedures of your facility, as well as federal and local regulations for “Sharps Disposal” (Fig. 5).

INSTRUCTIONS FOR USE: BLOOD COLLECTION

Aseptic technique, proper skin preparation and continued protection of the site are essential. Observe Universal Precautions on ALL patients.

PRECAUTION

Keep hands behind the needle at all times during use and disposal.

SET UP

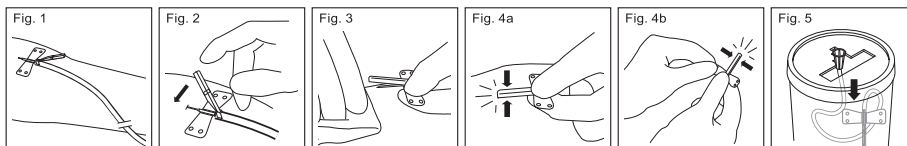
1. Open the package.
2. Connect a syringe, luer adapter, or other blood collection device and secure.

BLOOD COLLECTION

3. Flip the safety shield back away from the needle toward the tubing. Grasp wings.
4. Remove the needle protector. Caution: Care should be taken not to touch the needle.
5. Perform venipuncture following facility protocol (Fig. 1).

AFTER USE

6. Remove tape, if present, from wings.
7. Flip the safety shield forward towards the needle (Fig. 2). Grasp the wing with your thumb on top of the shield and your index finger under the wing. Completely remove the needle from the puncture site and apply digital pressure to the site using a sterile gauze pad (Fig. 3).
8. While maintaining the same grip on the device (per step 7 above), activate the safety feature by performing either of the following methods:
 - Recommended (one-handed) method:
(Reference OSHA 1910.1030 (d)(2)(vii)(B))
Slide your thumb forward over the shield while pinching your finger and thumb together (or press the safety shield against a hard surface such as a bedside table) until an audible click is heard and the safety feature is visually confirmed to be activated (Fig. 4a).
 - OR
 - Alternate (two-handed) method:
While keeping both hands behind the needle, grasp one of the wing tabs with your free hand to stabilize the device. (If necessary, reposition your original grip per step 7 being careful not to press against an exposed area of the needle.) Then, slide your thumb forward over the shield while pinching your finger and thumb together until an audible click is heard and the safety feature is visually confirmed to be activated (Fig. 4b).
- Caution: Needle must be carefully and firmly placed in the lock position of the safety shield.
9. Do not attempt to deactivate the safety device by separating the needle from the safety shield.
10. Dispose of used needles and materials following policies and procedures of your facility, as well as federal and local regulations for “Sharps Disposal” (Fig. 5).

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TERUMO MEDICAL PRODUCTS (HANGZHOU) CO., LTD.